



Molift Meditrans/Maxitrans



Molift Meditrans and Molift Maxitrans are ideal for transferring passive users between two level surfaces.

- **Reduces friction and makes the transfer easier for the caregiver.**
- **The padding ensures a comfortable and smooth transfer.**
- **Encourages the caregiver to adopt good posture during transfers.**

molift[®]
designed for life

Molift Meditrans and Maxitrans for transfers in a lying position

The Molift Maxitrans & the Molift Meditrans have a specially treated low-friction polyester on one side which facilitate slide transfers. When underneath the user they ensure comfortable transfers between two level surfaces, for example, between a bed and an X-ray table, an operating table or a shower table .



Molift Maxitrans & Molift Meditrans are placed under one side of the user by turning them from side to side. Placing the product underneath a sheet or draw-sheet will ensure an even better working posture for the caregiver.



The caregiver closest to the user can use the sheet to tilt the user onto the Molift Maxitrans or the Molift Meditrans, while the caregiver on the opposite side will then be able to easily pull the user across.



When removing Molift Maxitrans or Molift Meditrans, the user is rolled onto one side, then by pulling on the under side of the Molift Maxitrans or Molift Meditrans, it can easily be removed - as shown.

Specification

Material: Outside material: 100 % Polyester
Inside material: 100 % Polyester
Upholstery: 100 % Polyester
Non-allergenic material. Fire retardant material.

Measures: M-101070 Molift Maxitrans 23.5 x 74.5 inches
M-101065 Molift Meditrans 23.5 x 55 inches

Hygiene: The Molift Maxitrans & the Molift Meditrans can be washed at 157° F, or autoclaved at 265° F. Can be swabbed with water and soap or alcohol. Avoid use of chlorine.

Accessories: M-109010 GSI Disposable plastic covers, rolls of 150 pcs.



Molift - designed for life

At times with increasing demand for efficiency in the healthcare sector and a growing focus on patient needs, it is easy to neglect the needs of the caregiver. The need to give better help and caring avoiding heavy lifting situations, avoiding putting your own back at risk, without being completely exhausted after work and still have some surplus energy when the workday is over.

This is the insight that drives us at Molift. The company which was started 20 years ago by a man who through his own experience saw the need for lifting and moving patients in an effective and comfortable way – for both patient and caregiver.

We have since then delivered products that compliment the natural movement of the body, that is both simple and intuitive to use. Products designed to give the patient a better life in addition to providing the caregiver with real support and optimum work enjoyment.

This is the true meaning of; Molift – designed for life

molift

Molift Inc., 5008 W. Linebaugh Avenue, Suite 60, Tampa, FL 33624
Phone: 813-969-2213 Fax: 813-969-3954 toll free number 877-665-4387
www.molift.com