



Molift Handyswing Auto



The Molift Handyswing Auto composes of two fabric discs that are backed with low friction material. This allows the discs to turn on each other, making it easier for the user to turn when getting in or out of a car. Molift Handyswing Auto can also be used in the passenger and back seats.

- **The base is made of a non-slip material preventing it from sliding**
- **Neutral colors make it suitable for most cars**
- **Comfortable even on the longest of journeys**

molift[®]
designed for life

The Molift Handyswing Auto makes it easy to get in and out of a car

The Molift Handyswing Auto makes it easy for the user to access and exit the car.



The car seat is positioned so that the maximum amount of space is created for the user. The user sits as close as possible to the middle of the Molift Handyswing Auto as shown.



The legs are then lifted into the car. The Molift Handyswing Auto is designed to stay in the car seat during the journey - making it easier for the user to get out of the car at the end of the journey. Getting in and out of the car can be eased further by increasing the angle of the backrest.

Specification

Material:

Front side, towards user:	100 % Polyester, Noblesse
Rear side, towards seat:	100 % Woven Polyester, PVC coated
Inside material:	100 % Nylon
Upholstery:	100 % Polyester
Webbing	100 % Polypropylene

All non-allergenic material.

Measures: M-106015 Molift Handyswing Auto Ø 17.75 inches

Hygiene: The Molift Handyswing Auto can be washed at 139° F.
Do not use of chlorine.



Molift - designed for life

At times with increasing demand for efficiency in the healthcare sector and a growing focus on patient needs, it is easy to neglect the needs of the caregiver. The need to give better help and caring avoiding heavy lifting situations, avoiding putting your own back at risk, without being completely exhausted after work and still have some surplus energy when the workday is over.

This is the insight that drives us at Molift. The company which was started 20 years ago by a man who through his own experience saw the need for lifting and moving patients in an effective and comfortable way – for both patient and caregiver.

We have since then delivered products that compliment the natural movement of the body, that is both simple and intuitive to use. Products designed to give the patient a better life in addition to providing the caregiver with real support and optimum work enjoyment.

This is the true meaning of; Molift – designed for life

molift

Molift Inc., 5008 W. Linebaugh Avenue, Suite 60, Tampa, FL 33624
Phone: 813-969-2213 Fax: 813-969-3954 toll free number 877-665-4387
www.molift.com