



## Molift Handyswing Floor



**Molift Handyswing Floor is the perfect turning aid for a user who is able to weight bear. Its innovative design ensures that it will not turn until the caregiver initiates the turning movement.**

- **Facilitates a pivot transfer.**
- **The design ensures the user's stability.**
- **Together with Molift Support, Molift Handyswing Floor ensures a safe and comfortable transfer for both user and caregiver.**

## Molift Handyswing Floor facilitates a pivot transfer

**Molift Handyswing Floor** is manufactured from a specially-treated low-friction polyester fabric, and allows caregivers to rotate standing users easily.



By using the Molift Handyswing Floor, one caregiver can assist a user turn when in a standing position even when the user has little strength in their lower limbs (example: transfers between bed and wheelchair).



Control is achieved by the caregiver as they use their own foot to rotate the Molift Handyswing Floor.

## Specification

**Material:**

Outer fabric:	100% PA PVC
Inside fabric:	100 % Nylon
Foam:	100 % Polyester
Webbing	100 % Polypropylene

All non-allergenic material.

**Measures:** M-106020 Molift Handyswing Floor      Ø 15.75 inches

**Hygiene:** The Molift Handyswing Floor can be swabbed with water and soap or alcohol.  
Avoid use of chlorine.



## Molift - designed for life

At times with increasing demand for efficiency in the healthcare sector and a growing focus on patient needs, it is easy to neglect the needs of the caregiver. The need to give better help and caring avoiding heavy lifting situations, avoiding putting your own back at risk, without being completely exhausted after work and still have some surplus energy when the workday is over.

This is the insight that drives us at Molift. The company which was started 20 years ago by a man who through his own experience saw the need for lifting and moving patients in an effective and comfortable way – for both patient and caregiver.

We have since then delivered products that compliment the natural movement of the body, that is both simple and intuitive to use. Products designed to give the patient a better life in addition to providing the caregiver with real support and optimum work enjoyment.

This is the true meaning of; Molift – designed for life

***molift***

Molift Inc., 5008 W. Linebaugh Avenue, Suite 60, Tampa, FL 33624  
Phone: 813-969-2213 Fax: 813-969-3954 toll free number 877-665-4387  
www.molift.com