


## AM-SeatSling Appendix: Transfer from car (ED)


**REQUIRES 1-2 CAREGIVERS AND CEILING HOIST AT ED ENTRANCE  
(MAY REQUIRE A REMOTE CONTROL TO ACCESS HOIST)**

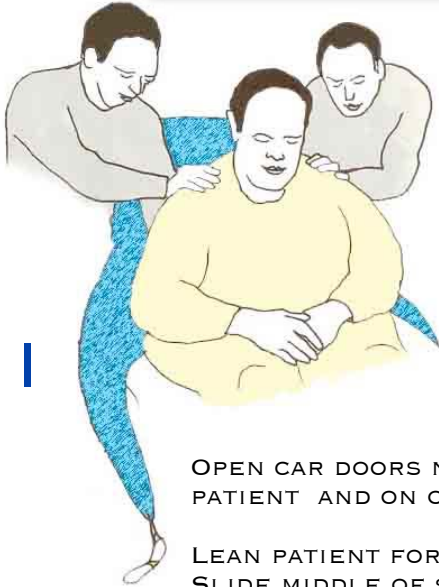

1


OPEN CAR DOORS NEXT TO PATIENT AND ON OPPOSITE SIDE.

LEAN PATIENT FORWARD.  
SLIDE MIDDLE OF SLING DOWN TO COCCYX.

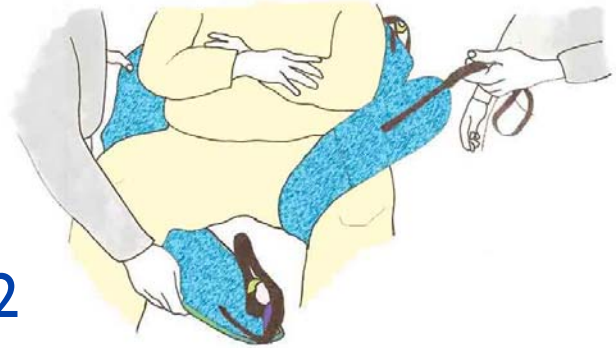

2

SLIDE EACH LEG WING UNDER PATIENT'S LEG JUST ABOVE THE KNEE--MAKE SURE BOTH WINGS ARE PULLED TIGHT SO THERE IS NO SLACK UNDER PATIENT.

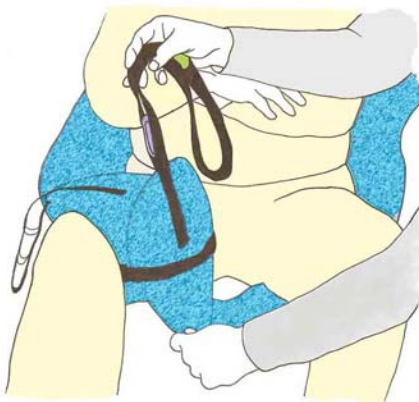

3

PULL EACH LEG WING UNDERNEATH PATIENT'S LEGS JUST ABOVE THE KNEE.

TAKE LEFT WING AND PULL IT THROUGH DIGNITY STRAP LOCATED ON THE RIGHT LEG WING.




4

ONE CAREGIVER GOES TO PARKED HOIST AND USES HANDSET TO LOWER HANGER BAR TO SHOULDER LEVEL.

WALK WITH HOIST TOWARDS PASSENGER SIDE.


**NOTE:** IF HANDSET IS TOO HIGH TO REACH, YOU WILL NEED TO USE A REMOTE CONTROL AVAILABLE AT THE FRONT DESK.

## AM-SeatSling Appendix: Transfer from car (ED)(cont.)

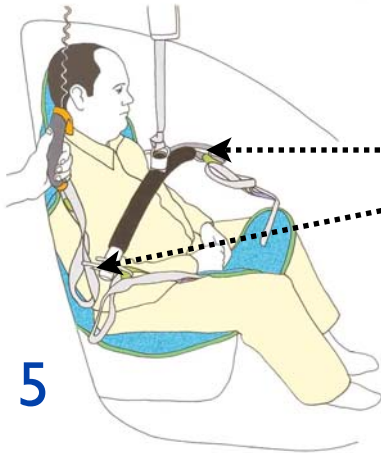

5

BRING HANGER BAR NEXT TO PATIENT SITTING IN THE CAR. LOWER HANGER BAR WITH HANDSET TO CHEST

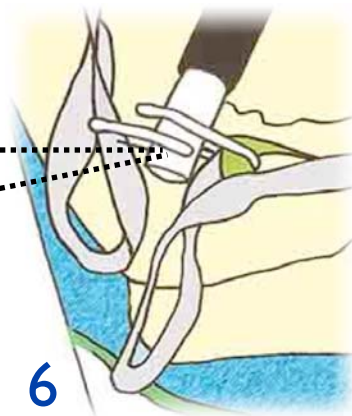

6

ATTACH LOOPS TO EITHER SIDE OF THE HANGER BAR AS SHOWN ABOVE.

USE THE BLACK LOOP FOR THE BACK STRAPS AND THE GREEN LOOP FOR THE LEG STRAPS.



7

BEGIN LIFTING PATIENT UNTIL WEIGHT IS OFF CAR SEAT.

THEN PIVOT PATIENT TOWARDS YOU AND PULL LEGS OUT OF THE CAR.



8

KEEP LIFTING THE PATIENT WHILE PULLING ON DIGNITY STRAP TO GET PATIENT OUT OF VEHICLE.




9

POSITION PATIENT ABOVE STRETCHER OR WHEELCHAIR.

BEGIN LOWERING PATIENT WHILE HOLDING TO DIGNITY STRAP FOR MORE STABILITY.

ONCE PATIENT IS ON STRETCHER, UNHOOK LOOPS AND RAISE HANGER BAR.

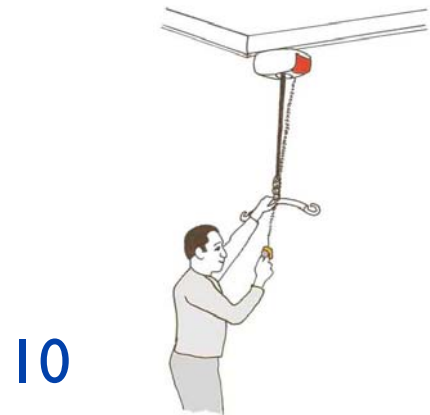

10

WALK HOIST BACK TO PARKING STATION.

MAKE SURE AMBER LIGHT COMES ON FOR CHARGING.

RAISE HANGER BAR TO ENSURE HEAD CLEARANCE.

HOOK HANDSET TO HANGER BAR.

IN SOME AREAS, YOU MAY NEED REMOTE CONTROL TO CONTINUE RAISING HANGER BAR.