

**TRIM INDICATES SIZE/WEIGHT\***

<b>S</b>	31-75 LBS	14-34 KG
<b>M</b>	75-145 LBS	34-66 KG
<b>L</b>	145-220 LBS	66-100 KG
<b>XL</b>	220-440 LBS	100-200 KG
<b>XXL</b>	440-800 LBS	200-363 KG

*\* IF IN DOUBT STEP DOWN ONE SIZE*



1


ELEVATE HEAD OF THE BED. PLACE SLING BEHIND PATIENT, WITH **LABEL** ON THE OUTSIDE.

THE **LEG WINGS** SHOULD COME ALONG THE SIDE OF THE PATIENT



1b-2b

**ALTERNATIVE:**  
WHEN BED CANNOT BE ELEVATED:  
USE LOGROLLING TO PLACE SLING  
UNDER PATIENT.

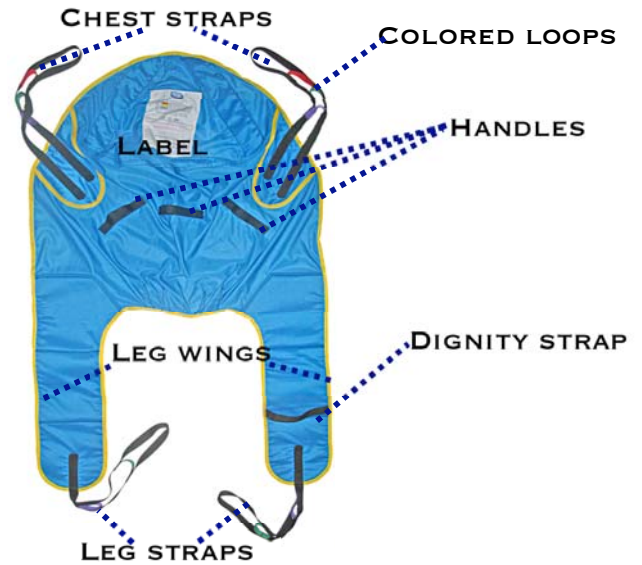


2




LEAN PATIENT FORWARD. SLIDE CENTER OF SLING DOWN TO THE COCCYX AREA

3

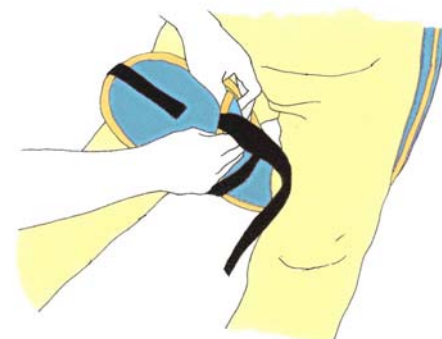


PULL EACH **LEG WING** UNDERNEATH PATIENT'S LEGS JUST ABOVE THE KNEE. TAKE LEFT WING AND PULL IT THROUGH **DIGNITY STRAP** LOCATED ON THE RIGHT

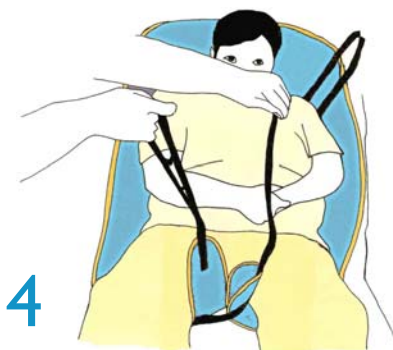

4

PULL LEG STRAPS TO STRAIGHTEN FABRIC UNDER PATIENT.

DO NOT CRISSCROSS THE **LEG WINGS**

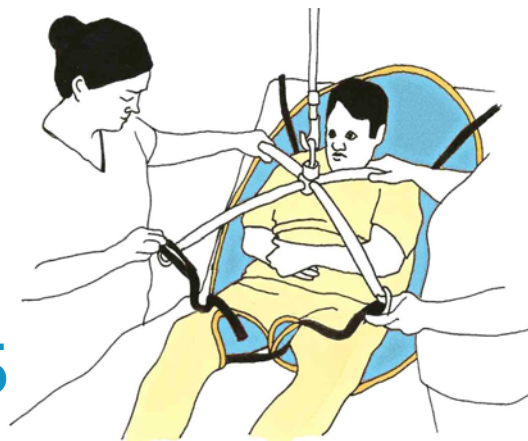

5

ATTACH STRAPS TO HANGER BAR

FOR SEATED POSITION: SHORT LOOP FOR **CHEST STRAPS** AND LONG LOOP FOR **LEG STRAPS**

FOR SEMI-RECLINED TRANSFER: LONG LOOP FOR **CHEST STRAPS** AND SHORT LOOP FOR **LEG STRAPS**



6

AFTER LIFTING, MOVE PATIENT BY HOLDING THE SLING OR THE **HANDLES** LOCATED BEHIND THE SLING

NOTE: PICTURE SHOWS SEATED TRANSFER



7

TO LOWER PATIENT INTO CHAIR, PUSH GENTLY FORWARD ON THE **LEG WINGS** TO ENSURE PATIENT'S BUTTOCKS ARE IN CONTACT WITH THE BACK OF THE SEAT.