


AM-TSL Log Roll/ Repositioning: Hip/Knee Precautions

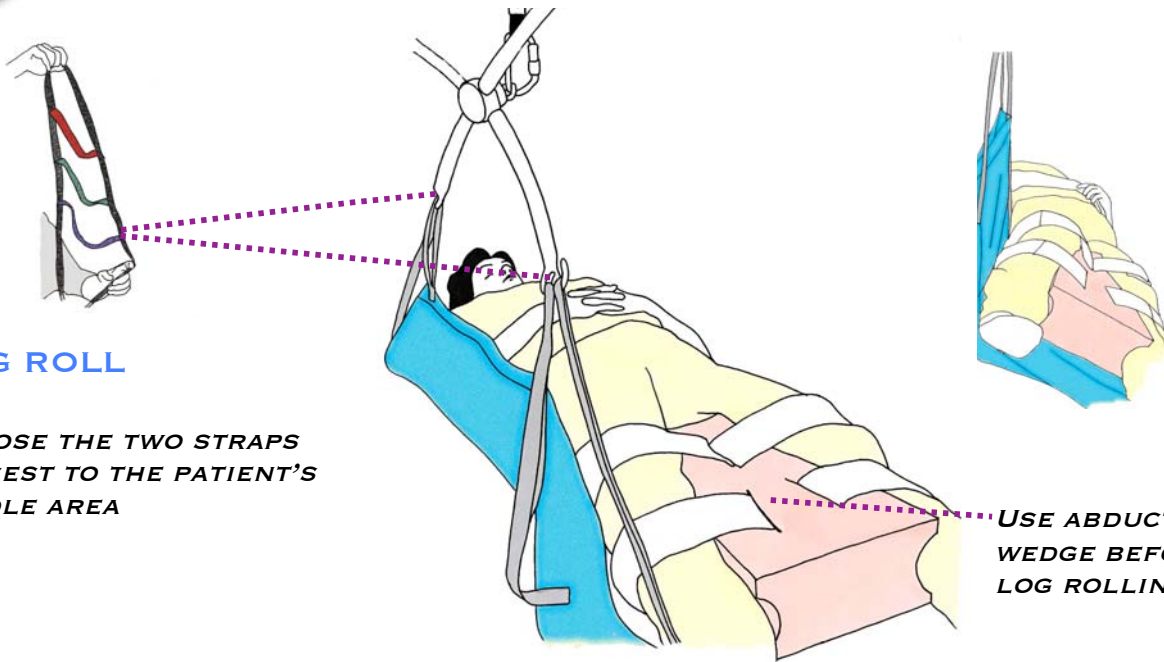


LOG ROLL

*CHOOSE THE TWO STRAPS
CLOSEST TO THE PATIENT'S
MIDDLE AREA*

*USE ABDUCTOR
WEDGE BEFORE
LOG ROLLING*

REPOSITION

*PLACE ABDUCTOR WEDGE
BETWEEN LEGS AND SECURE
BEFORE REPOSITIONING OR
LIFTING*

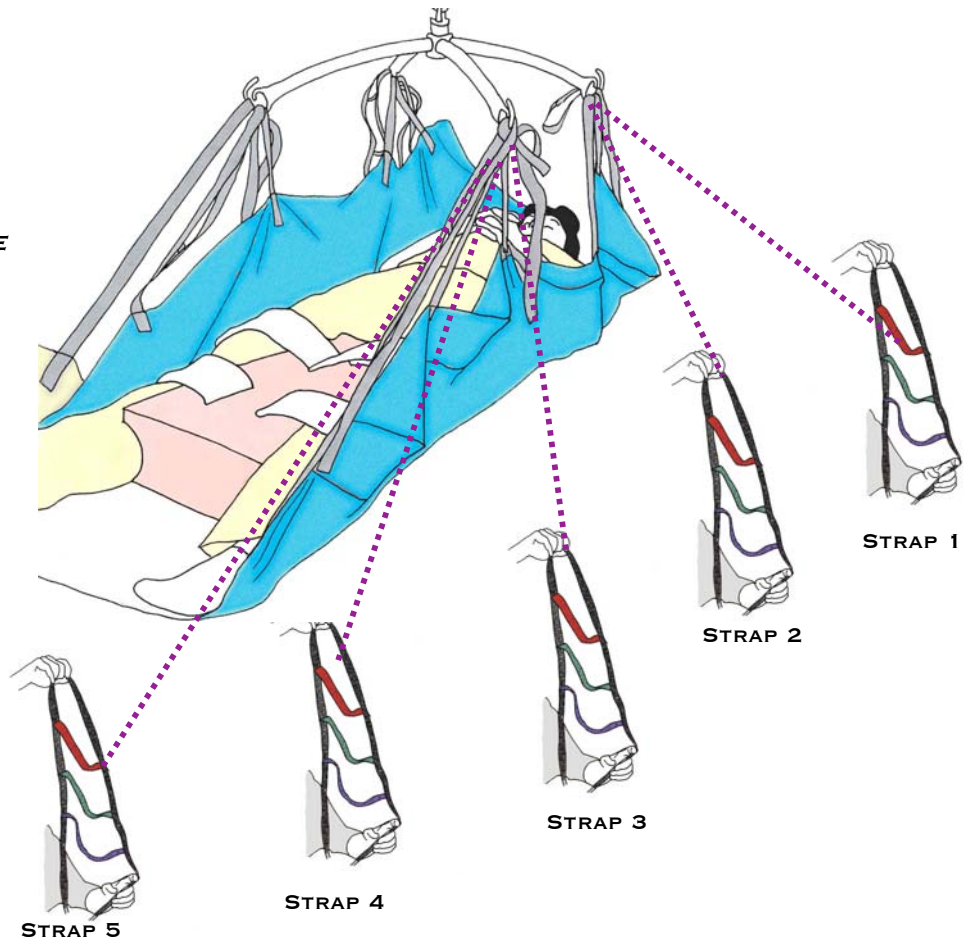

STRAP 1

STRAP 2


STRAP 3

STRAP 4

STRAP 5

800 | 273 | 5749

www.alphamodalities.com

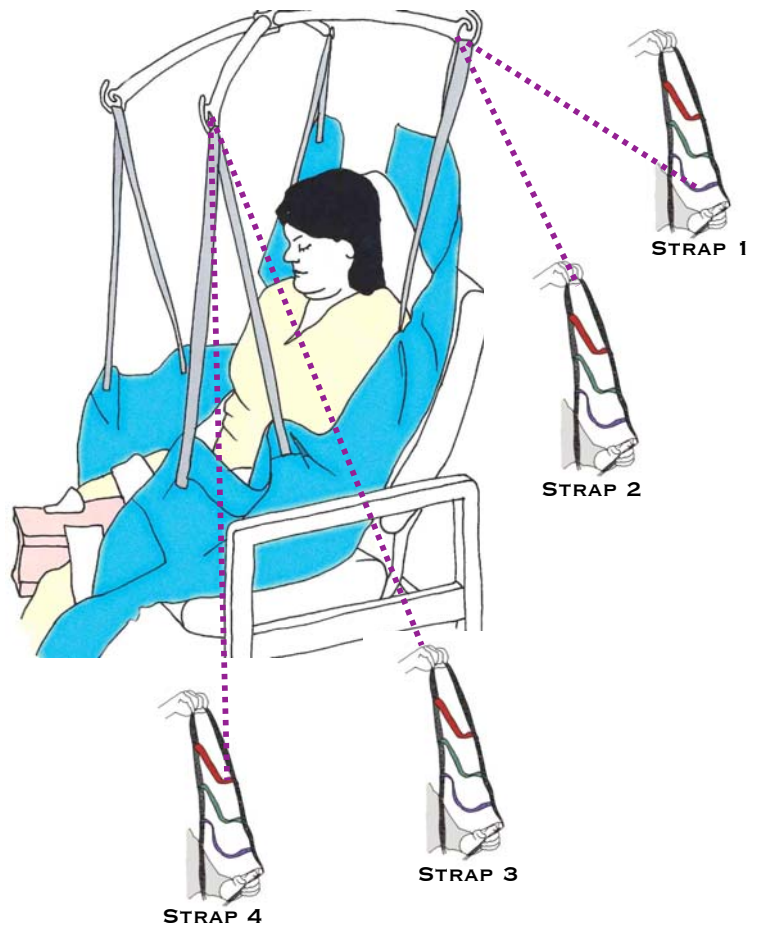

TRANSFER TO GERI-CHAIR

**PLACE ABDUCTOR WEDGE
BETWEEN LEGS AND SECURE-
ATTACH LOOPS TO LIFT PATIENT
IN SEATED POSITION.**

**WHEN LOWERING PATIENT INTO
CHAIR, SUPPORT THE ABDUCTOR
WEDGE WITH A HAND UNTIL
PATIENT IS FULLY SEATED- THEN
REMOVE.**

