

AM-TubeSlider



Tab Hook color indicates size


S	40 X 28"	102 X 71 cm
M	48 X 48"	122 X 122 cm
L	52 X 78"	132 X 198 cm

IT IS RECOMMENDED TO PLACE A CHUCK OR DRAW SHEET BETWEEN PATIENT AND TUBE SLIDER.




1

TO PLACE TUBE SLIDER UNDER PATIENT, GRAB TWO CORNERS AND FAN FOLD ON BOTH SIDES.

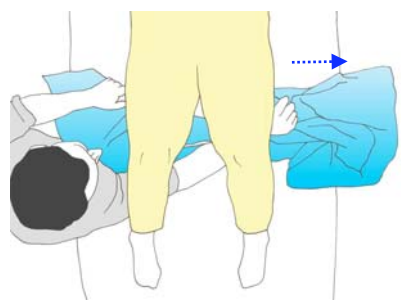


2

BEND PATIENT'S KNEES WITH THEIR HEELS PUSHING INTO BED. THEN PUSH DOWN AND SLIDE TUBE UNDER LEGS ACROSS TO THE OTHER SIDE.

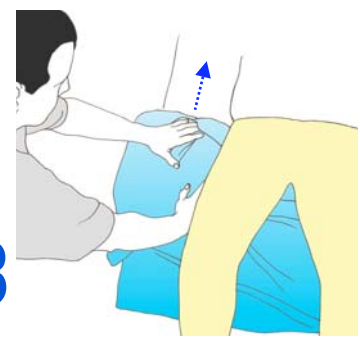

3

PUSH TOP OF THE SLIDER DOWN AND SLIDE IT UP THE BED ON BOTH SIDES OF THE PATIENT--USE TWO CAREGIVERS WHEN POSSIBLE.

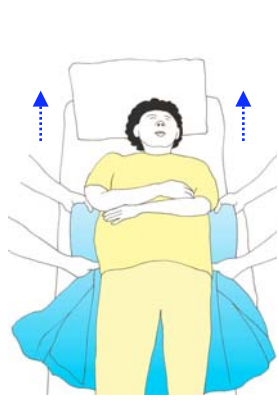

4

ONCE SLIDER IS EVENLY SPREAD UNDER PATIENT, GRAB TOP OF SHEET ON BOTH SIDES, CLOSE TO THE PATIENT'S BODY. PUSH PATIENT UP TOWARDS THE HEAD OF THE BED WITHOUT LIFTING.



5



TO REMOVE SLIDER FROM UNDER PATIENT, SLIDE ARM INSIDE TUBE AND UNDER PATIENT'S LEGS. GRAB BOTTOM FABRIC LAYER ON OPPOSITE SIDE. PULL BOTTOM LAYER TOWARDS YOU UNTIL THE TUBE SLIDER COMES OUT FROM UNDER THE PATIENT .