

- 1 LOG ROLL THE PATIENT TO PLACE THE HOVERMATT UNDERNEATH.**

USE A SHEET BETWEEN THE PATIENT AND THE HOVERMATT TO PREVENT SOILING.

NOTE: LABEL SHOULD BE FACING UP AT FOOT END.




- 2 ATTACH PATIENT SAFETY STRAPS**

- 3 PLUG ELECTRIC CORD INTO OUTLET**

ATTACH AIR SUPPLY HOSE AT EITHER FOOT END OF HOVERMATT®.

MAKE SURE THE SNAP AND VELCRO FLAP ARE ATTACHED.

- 4 PUSH GREY BUTTON ON AIR SUPPLY TO TURN IT ON AND INFLATE THE HOVERMATT.**

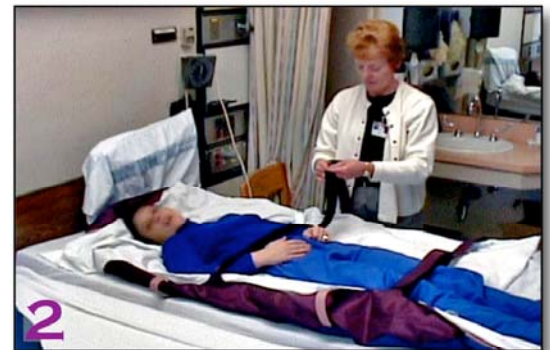

5 ADJUST WORKING SURFACE TO PROPER HEIGHT. PUSH THE TWO WORKING SURFACES TOGETHER. LOCK THE WHEEL BRAKES.



6-7 GRASP THE SIDE HANDLES AND **GENTLY** PULL THE PATIENT ON AN ANGLE, **HEAD OR FEET FIRST**, UNTIL PATIENT IS CENTERED IN THE RECEIVING SURFACE.



DO NOT USE THE PATIENT STRAPS TO PULL THE HOVERMATT®.

AFTER COMPLETING TRANSFER, TURN OFF THE AIR SUPPLY AND DISCONNECT THE HOSE FROM THE HOVERMATT®.



8 IF HOVERMATT® IS LIGHTLY SOILED AFTER USE, WIPE IT WITH SPRAY DISINFECTANT OR SANIWIPE.

IF THE HOVERMATT® IS HEAVILY SOILED, BAG SEPARATELY AND FOLLOW HOSPITAL PROTOCOL FOR WASHING.



NEVER LEAVE A PATIENT UNATTENDED ON AN INFLATED MATTRESS

NOTE: THE HOVERMATT® IS RADIOLUCENT AND MAY BE LEFT UNDER THE PATIENT DURING IMAGING

