



## OMEGA 300 FLOOR LIFT OPERATION

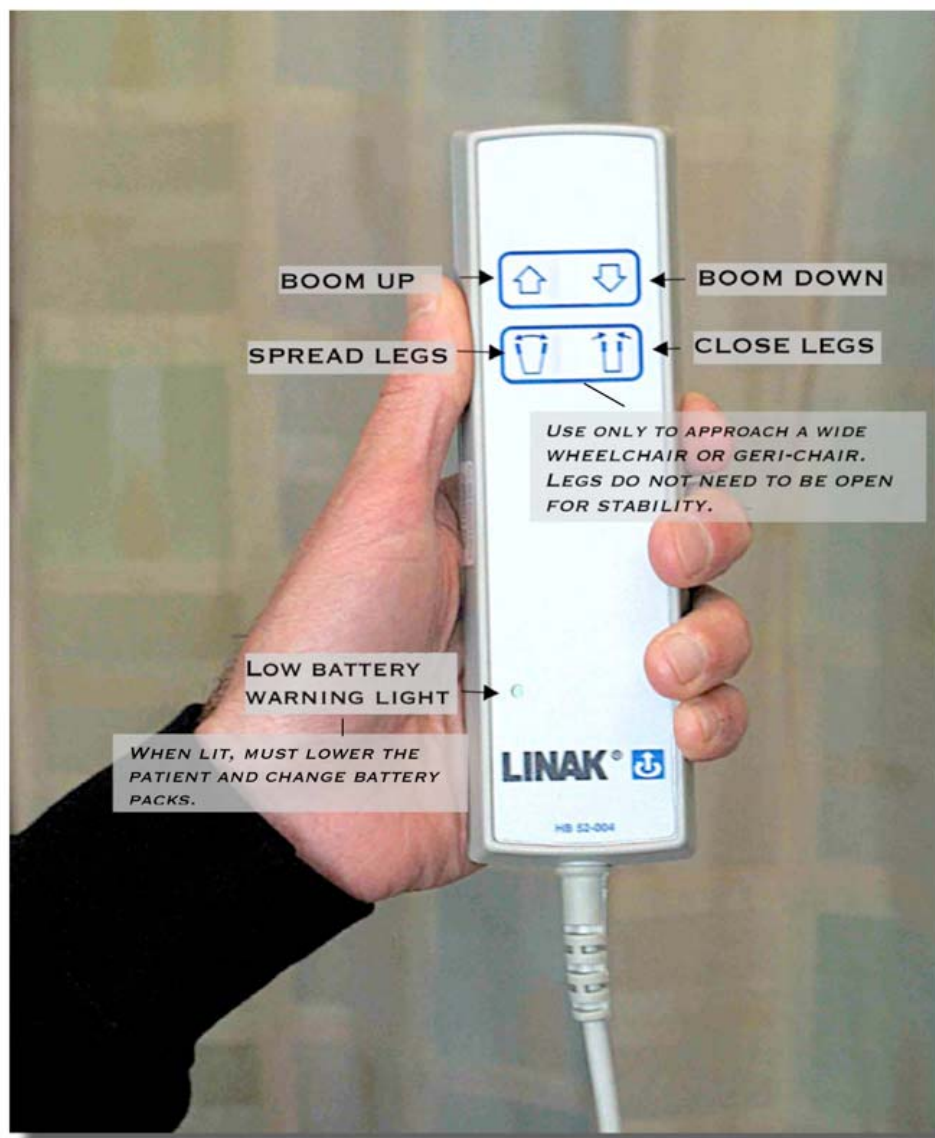



## OMEGA 300 FLOOR LIFT OPERATION

THE BATTERY PACK NEEDS TO BE RECHARGED DAILY.

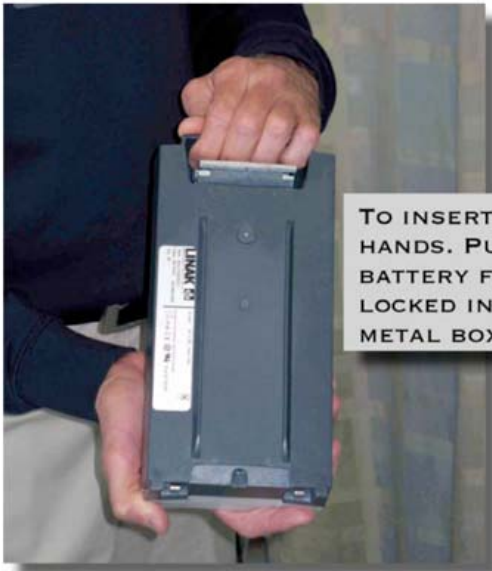

TO INSERT THE BATTERY PACK, HOLD IT WITH TWO HANDS. PULL UP THE UPPER BACK HANDLE. PULL THE BATTERY FORWARD ONCE SEATED TO MAKE SURE IT IS LOCKED IN PLACE. INSERT IT ON THE TOP OF THE METAL BOX AS SEEN BELOW.

ONCE THE BATTERY IS INSERTED CHECK THE CHARGING LEVEL WINDOW. DO NOT OPERATE THE LIFT IF THIS WINDOW SHOWS ONLY ONE BAR.



## OMEGA 300 FLOOR LIFT OPERATION

### EMERGENCY STOP

PRESS THE RED KNOB IN UNTIL IT CLICKS. THIS WILL SHUT OFF ALL LIFT OPERATIONS



TO RESET THE LIFT, TURN THE RED KNOB IN THE DIRECTION OF THE ARROW ON THE KNOB UNTIL IT CLICKS OUT. THE LIFT IS NOW READY TO USE.

### EMERGENCY LOWER

1. IF THE LIFT SYSTEM SHOULD FAIL: CHOOSE THE BED OR CHAIR IN WHICH YOU WANT TO LOWER YOUR PATIENT--MOVE THE PATIENT TO THAT AREA.
2. PLACE A FINGER IN THE RED "U" BRACKET ON THE HOIST MOTOR AND PULL UP--PULL UP FOR A FEW SECONDS AT A TIME, IT DROPS THE PATIENT QUICKLY. IF YOU RELEASE, IT WILL LOCK IN PLACE

