




Alpha Modalities LLC  
 T | 800.273.5749  
 F | 206.374.2476  
[www.alphamodalities.com](http://www.alphamodalities.com)

# SINGLE PATIENT USE (SPU) SLING INSTRUCTIONS

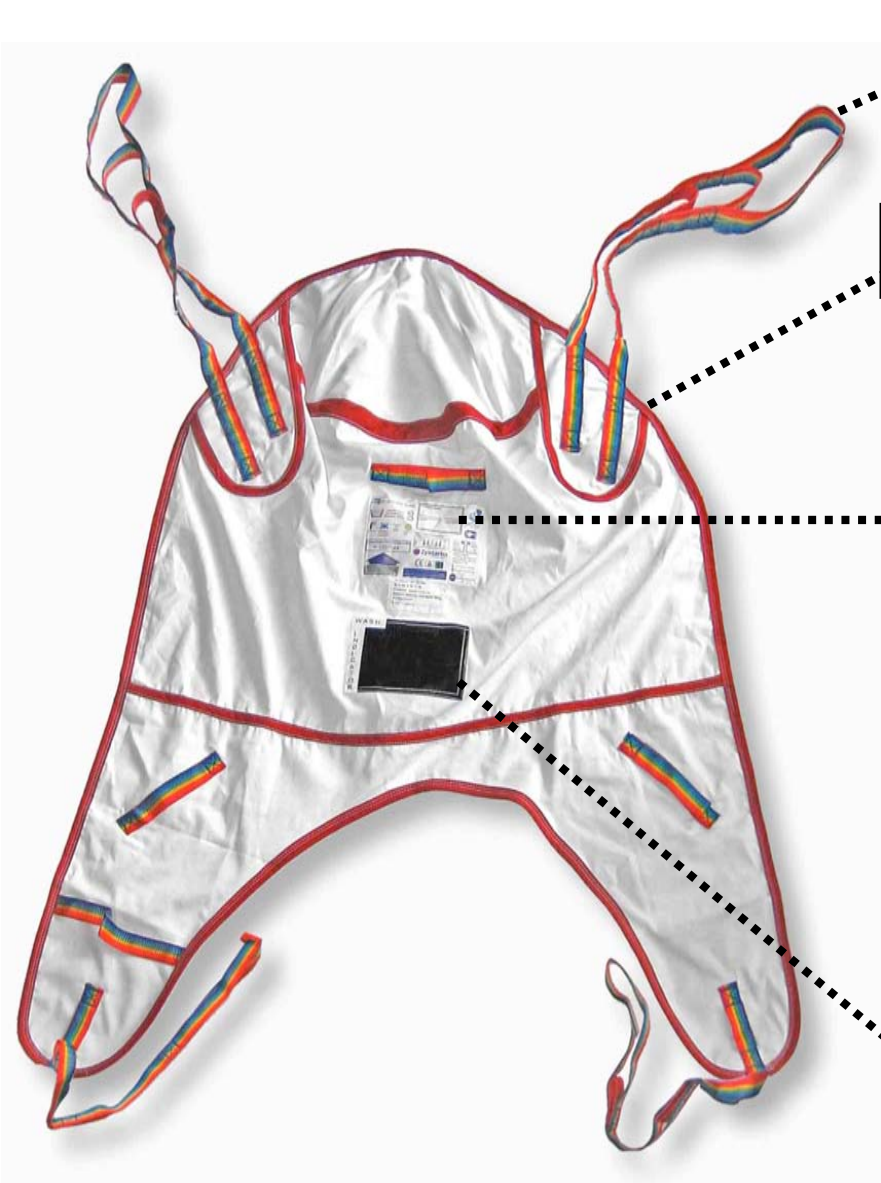

SPU SLINGS ARE IDENTIFIED BY RAINBOW STRAPS

TRIM COLOR INDICATES SIZE/WEIGHT →

BEFORE FIRST USE WRITE PATIENT INFORMATION ON LABEL →  
DO NOT USE WITH OTHER PATIENTS


**CLEANING INSTRUCTIONS:**  
 SPOT CLEAN IF SOILED.  
DO NOT WASH  
DO NOT USE SLING IF WASH INDICATOR LABEL IS VISIBLE. →  
 IT MEANS THE SLING HAS BEEN WASHED

S	31-75 LBS	14-34 KG
M	75-145 LBS	34-66 KG
L	145-220 LBS	66-100 KG
XL	220-440 LBS	100-200 KG
XXL	440-800 LBS	200-363 KG

<b>PATIENT NAME</b>	
PATIENT ID #	
<b>DATE</b>	<b>LOCATION</b>
FIRST USE	ROOM #

WASH

INDICATOR



Patient Specific Sling (Disposable) Sling  
**DO NOT USE THIS SLING**  
 THIS SLING HAS BEEN WASHED